

Doctors say the **3MD** can be used to improve the appearance of:

- Aging, loose, and dull-looking skin
- Vertical lip lines
- Vitiligo*
- Hyper-pigmentation
- Enlarged/Dilated pores
- Stretch marks
- Scars (acne, surgical and burns)
- Alopecia*

*when used in conjunction with topical treatment



B & A

REJUVENATION RESULTS*
COURTESY DR. MARZENA LORKOWSKY-PRECHT



TREATMENT OF TRAUMA SCAR*
COURTESY OF DR. HASAN EL-FAKAHANY



*Results may vary. Treatment frequency / number of sessions will be determined by the clinician for each individual patient.

Make an appointment with your clinician today!

Check with your clinician before proceeding with a microneedling procedure if you are: Pregnant, using blood thinners, such as warfarin, or if you are susceptible to keloiding skin. Do not treat over keloid scars, active problematic skin, such as severe solar keratosis, raised moles/warts, Rosacea, Eczema or Psoriasis.

The provider of this service has been informed by DP Derm LLC of local laws which may apply to the operation and use of the 3MD™ device. DP Derm LLC hold no responsibility for treatment outcomes. Individual results may vary. To be used only as directed. INTD3M3004.10/15

For more information go to:

DpDerm.com



3MD
Advanced Microneedling

Erase Your Skin's Past


WHY CHOOSE 3MD ADVANCED MICRONEEDLING?

The 3MD™ Advanced Rejuvenation system from DP Derm utilizes fractional micro-channeling to pierce the skin's surface at a 90 degree angle. During the procedure, the sterile tip of a 3MD™ microderm cartridge gently glides over the skin, creating millions of fine channels. These channels subsequently help deliver topically applied formulas deeper into the skin.

3MD™ systematically creates micro-channels in the skin and the body naturally responds by launching processes to repair the physical damage. The result is an increased production of normal woven collagen, similar to the reaction to fractional laser treatments, but without the side effects and downtime.

When compared to traditional dermal rollers, 3MD™ causes less epidermal (skin) damage, reducing levels of pain and discomfort. As a result, this device is a safer, more comfortable, and more effective micro-needling treatment (see image below). Sterile single use cartridges eliminate risk of cross-contamination, whilst the adjustable settings allow for a treatment that is fully tailored to your individual needs. The design of the device also allows for it to be used in hard to reach areas such as around the eyes, eyelids, earlobes, neck, lips and on the nose.

Your clinician is best suited to assess your skin and treatment area, as well as recommend a treatment plan that will be most beneficial for you.

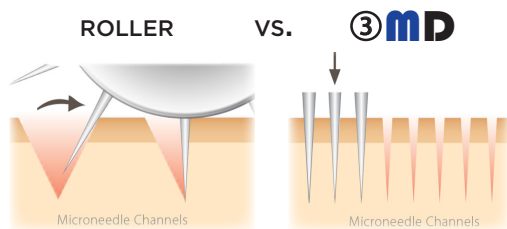

Dermal rollers operate via a rolling motion, causing lateral openings; 3MD™ creates micro-channels that are 100% vertical, which reduces pain and discomfort.

FAQ's

How many treatments are recommended for rejuvenation, acne scarring and stretch marks? (General guide)

Rejuvenation:	4-6 treatments	4 weeks apart
Acne scarring:	6 treatments	6 weeks apart
Stretch marks:	4-8 treatments	4-6 weeks apart

How should I prepare my skin for a 3MD™ Treatment?

For optimal results, an appropriate regime of DP Dermaceuticals™ should be determined by your clinician, to encourage the skin to strengthen (especially for thin or delicate skin areas). Continue to use DP Dermaceuticals™ recommended by your clinician after the treatments for a period of 6 weeks. The DP Dermaceuticals™ range is suitable for ongoing home care.

How long should it take before I see results?

Typically, you will see results after the first treatment. Lasting and more significant results will become visible after 4 to 6 treatments (spaced 4-6 weeks apart). The condition of your skin will continue to improve over the next 6-12 months after a course of treatments, especially when combined with the recommended post-treatment care.

Is the treatment painful?

3MD™ treatments are relatively painless. Deeper treatments may require a topical anesthetic cream to be applied for 15-40 minutes before the procedure. This will render the treatments virtually pain-free.

Is there any downtime after treatments?

Generally no, but you may experience facial redness for 12 to 48 hours after the treatment. Most people are able to return to normal daily activities immediately after the treatment. Application of a breathable camouflage will seal the skin and cover any redness for an immediate return to work -- try DP Dermaceuticals Cover Recover!

How soon after filler/muscle relaxant injections can I have a micro-needling treatment?

While there is no clinical evidence that 3MD™ Treatments counteract with any particular injectables, it is important to let your clinician know if you have had an injectable treatment, or are planning on having one done in the near future. Your clinician will be able to advise you on when to schedule your treatment appropriately.

HOW TO CARE FOR YOUR SKIN BETWEEN TREATMENTS

STEP 1 Advanced Resurfacing

Your clinician will apply a suitable topical agent to compliment your 3MD™ Treatment. Between treatments, continue using recommended products as directed to maintain results. DP Dermaceuticals MicroDerm™ Exfoliant* is one option for gentle but effective resurfacing between treatments.

STEP 2 Hydrate and Strengthen

For best results, it's important to keep your skin hydrated and protected from the harsh environment to make it healthy and strong. DP Dermaceuticals Vitamin Rich Repair™ Cream* was specially created to enhance the results of your treatment, by promoting intensive rebuilding and repair of the skin pre and post treatment. If you will be exposed to the sun, always wear appropriate sun protection.

STEP 3 Cover, Recover and Protect for Beautiful Skin

It is unlikely that you will experience any side effects after your treatment. However, if you do experience redness or sensitive skin, you may use a breathable post-procedure camouflage makeup. This will allow you to get back to your everyday activities with no one suspecting that you have even had a treatment. In addition, camouflage makeup coverage can also work as sun protection for the skin. Try DP Dermaceuticals Cover Recover™ formula!

*Ask your treatment provider about the full range of DP Dermaceuticals™.

